

petMAPTM graphic Quick Reference Card



figure 1



figure 2



figure 3

1. Apply a Cuff

- Select measurement site (forearm or tail preferred, hind foot - least preferred).
- Select proper size cuff (based on measurement site, animal's size and cuff index markings).
- Apply the cuff snugly.

2. Set up the petMAPTM

- Power on by pressing the Power/Page button (the left top button, see figure 1).
- Select species and cuff site location by pressing the Multi-Function button (the right top button, see figure 1).
- Attach cuff hose securely to petMAP's cuff connector.

3. Take Readings

- Inflate the cuff by gently squeezing the bulb. Note the cuff pressure on the display. Inflation pressure should be 40 - 50 mmHg higher than the expected systolic pressure. If systolic pressure is unknown, inflate to ≈ 220 . Do not over-inflate.
- Allow petMAP to deflate and publish the readings.
- In the presence of motion artifact, abort the reading by releasing the pressure using the red trigger (see figure 1). Once the pressure is released and the patient is calm, close the red trigger and begin another reading.
- 3 - 5 determinations are suggested or until consistent readings are achieved.

4. Review and Chart Data

- For a graphical representation of results, press the Power/Page button once (see figure 2).
- For a tabular listing of results, press the Power/Page button again (see figure 3).

5. Cycle Power

- To turn the petMAP OFF either press both top buttons simultaneously or press and hold the Power/Page button for 4 seconds.

Note: Cycling power begins a new session. The previous session's NSV is displayed on power up and the previous session data is available until the cuff is inflated to begin the next session.

This Quick Reference Card should be used in combination with the product's Operator's Manual. If the user has questions not addressed in the Operator's Manual, please refer to www.petmap.com or contact Customer Service at (800) 231-6370 or (813) 289-5555.